



SafeMD Suture Smart Cart  
Mobile Operating Table  
Disinfection Instructions

1. Wear appropriate personal protective equipment
2. Remove and appropriately dispose of all hazardous sharp materials
3. Remove convenience tray and used procedure materials and dispose in designated waste container
4. Brush, wipe, wash all visible material from work surface
5. Place perforated work surface in Ortho-phthalaldehyde (Cidex OPA). Submerge completely for 5 minutes, remove, and wash with water, clean and dry.
6. Spray Cidex OPA on work surface area and irrigation capture area. Let stand 5 minutes. Wash with water and dry.
7. Remove irrigation capture bottles. Dispose of all liquids in and appropriate liquid waste system.

\* The mobile operating table is classified as semi-critical item under the Spaulding approach to disinfection of patient- care items and equipment.